vivocare

health enjoyment

Model: **STEAMY**

Compressor Nebulizer
Kompresörlü Nebülizör

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EN Instruction Manual

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EN

**Read this instruction
manual carefully before use**

TIEC Symbols

Caution Attention or Consult accompanying documents.....	
Type BF Equipment.....	
Alternating Current.....	
Double Insulation.....	
Indoor use only.....	

Important Safeguards

Caution:

1. Follow the instruction of your physician to operate this unit.
2. This product is a nebulizer for the inhalation of medical aerosols and is suitable for children used solutions. Use only the type and amount of medication prescribed by patient's doctor.
3. This product is intended for aerosol therapy only. And other use is not recommended.
4. Do not untwist the nebulizer during operation.
5. Do not pour more than 10ml of solution into the nebulizer.

Danger: To Reduce the risk of electrocution.

6. Always unplug the unit immediately after using.
7. Do not use while bathing.
8. Do not place or store the unit where it can fall or be pulled into a tub or sink.
9. Do not place or drop into water or other liquid.
10. Do not reach for a product that has fallen into water .Unplug immediately.

 Warning: To Reduce the risk of burns, electrocution, fire or injury

1. Electrical Shock Hazard—do not remove the cabinet or open the cover.
2. Disconnect the power cord from the electrical outlet before cleaning or servicing.
3. Do not place this equipment near hot, sparking or burning objects.
4. Do not use oil or grease on or near this device.
5. Turn the unit off when it is not in use.
6. Keep the cord away from HEATED or HOT surfaces.
7. NEVER drop or insert any object into any opening.
8. NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked.
9. Avoid operating in wet or damp locations.
10. Unplug the unit before filling the nebulizer.
11. When using this product near TV, microwave oven, pulse telephone X ray or other strong electric field, it will be disturbed, suggested to be far away from these device measuring.

 Warning: To reduce the risk of infection:

1. This Nebulizer kit is intended for both adult and pediatric patient use.
2. Cleaning of the nebulizer is recommended after each aerosol treatment. disinfecting is recommended once a day. Please follow cleaning and disinfecting instructions in this manual.

This unit is not suitable for use in anaesthetic breathing systems

This unit is not suitable in suspension or high viscosity form. In such cases information should be sought from the drug supplier

Introduction

This compressor nebulizer is designed to deliver the prescribed medication solution to treat patient respiratory disorders, such as asthma, allergies and bronchitis. The nebulizer converts the medication solution into an aerosolized mist which is inhaled by the patient through the mouthpiece or masks. Read this manual thoroughly before using the nebulizer and save for future reference

Specification

Electrical Requirements	120V/60Hz, or 230 V / 50Hz
Power Consumption	Below 65W or 60W
Rated Current	Below 1.5A or 0.70A
Sound Pressure Level	58 dB(A)
Max.Comp.Pressure	35 Psi to 50psi (241 kpa to 345 kpa)
Neb.Operating.Pressure	9~16Psi (62kpa to 110kpa)
Liter Flow Range	5-8 L/Min
Nebulization Rate:	≥0.25 ml / Min
Particle size	3 microns
Maximum nebulizer solution capacity	6ml
Maximum residual medicine volume	0.5 ml
MMAD:	4um
Average Mobilization Rate:	Min 0.32ml / Min
Remain Amount:	0.15ml
Operating Temperature Range°C	10°Cto 40°C(50°F to 104°F)
Operation Humidity Range:	10%~95%RH
Storage Temperature Range°C	-20°Cto 70°F(-4°C to 158°F)
Storage Humidity Range:	10%~95%RH
Dimension(LxWxH):	Length 280mm x Width 165mm x Height 90mm
Products Weight:	1.8 kgs
Standard Accessories:	Compressor Nebulizer Machine : 1pcs
	Adult Mask: 1pcs
	Air Filter:1pcs
	Durable Air Tube:1pcs
	Medical Cup 6CC:1pcs
	Mouth Piece: 1pcs
	Nosepiece: 1pcs

Protection against electric shock

☐ - Classified as Type II

⚠ - Type BF applied parts: Mouthpiece, mask

Degree of safety in the presence of flammable anesthetics or oxygen

-No AP/APG (not suitable for use in the presence of flammable anesthetics or oxygen)

Mode of operation - Continuous

IP21 - Protection against harmful in gress of water is Ordinary

Description

1. Nebulizer machine

2. Nebulizer cup

3. Air tube

4. Mouthpiece

5. Air filters

6. Adult mask

7. Pediatric mask

8. Nosepiece

Nebulizer: Machine which converting liquid medicine to aerosol

Nebulizer cup: Cup for liquid medicine

Air tube: Connection between nebulizer and nebulizer cup

Adult mask: Mask for breathing medicine through mouth and nose for adults

Pediatric mask: Mask for breathing medicine through mouth and nose for children

Mouthpiece: Accessory for breathing medicine through mouth

Nosepiece: Accessory for breathing medicine through nose

Air filter: Piece to filter the air coming from outside into nebulizer

Operation Instructions

1. Place compressor on a stable, sturdy and flat surface so that the unit can be easily reached when you are seated.
2. Pull out the power cord from the base of the unit ,and make sure that the unit is in the“ off” (O) position by pressing on the left side of switch.
3. Plug power cord into a wall outlet.
4. Connect one end of the nebulizer tubing into the air outlet connector.
5. Add the prescribed solution through the opening on cap using an eye dropper or pre-measured dose container.
And the marking “6cc” marked on the nebulizer cup is the maximum level
And the medication can not be filled beyond the maximum level
6. Assemble mouthpiece and T-mouthpiece, and insert into the top of the nebulizer cap. If an aerosol mask is used ,connect the bottom of mask to the top of nebulizer cap.
7. Connect tubing to nebulizer air inlet connector.
8. Turn the power on to start the unit.
9. Implement the treatment by placing the mouthpiece between the teeth. Inhale and exhale through the mouthpiece.
10. If an aerosol mask is used, place the mask over the mouth and nose.

Treatment procedure

- Wash your hands with soap and water, and dry them with a clean towel.
 - Measure your medicine.
 - o Always wash your hands before preparing the treatment.
 - o Always use clean measuring devices (eyedroppers or syringes).
 - o Use a separate measuring device for each solution used in your treatment.
 - o Measure your medicines exactly as you have been instructed.
 - o Wash your hands after measuring each solution used in your treatment.
1. Remove the top part of the nebulizer cup.
 2. Place the medicine in the bottom of the nebulizer cup.
 3. Attach the top portion of the nebulizer cup, and connect the mouthpiece or face mask to the cup. Connect the tubing to the nebulizer and compressor.
Turn on the compressor with the on/off switch.
 4. Once you turn on the compressor, you should see a light mist
 5. If you are using a mouthpiece, place it in your mouth and seal your lips tightly around it.

6. If you are using a face mask, position it comfortably and securely on your face.
- Inhale slowly and deeply. Hold your breath for a count of five and exhale slowly.
 - Continue to inhale slowly and deeply for about five to 10 minutes or until the medicine is gone.
 - If you become dizzy, shaky, or feel an increased heart rate, stop your treatment and rest for about five minutes. Then continue the treatment, but breathe more slowly. If these symptoms continue, call your health care provider.
- Turn the compressor off and unplug it.
 - Take several deep breaths and cough. Continue coughing and try to clear any secretions you might have in your lungs. Cough the secretions into a tissue and dispose of it properly.
- Wash your hands with soap and water, and dry them with a clean towel.

Cleaning

1. Turn the power off and unplug from the wall outlet .
2. Remove the tubing from the air inlet.
3. TO CLEAN: Disassemble mouthpiece, T-mouthpiece, nebulizer cup, chamber and baffle, and wash this items in hot water with dish-washing detergent Rinse these items thoroughly to remove the detergent and let air dry.
4. TO DISINFECT: Mix one part of white vinegar with 3 parts hot water in a clean container. Submerge mouthpiece, T-mouthpiece, nebulizer cap, chamber and baffle for half an hour in the solution. Remove from solution and let air dry.
5. There is no need to clean the tubing. If necessary wipe the surface regularly.

Note:

Nebulizer accessories including child mask , mouthpiece, nosepiece, nebulizer cup ,PVC tube , can be recycle used in 5 times.

Lifetime of nebulizer is 3 years

CAUTION: The nebulizer must be changed if it gets clogged.

CAUTION: The nebulizer and accessories must not be boiled.

Maintenance

1. The filter of air compressor should be replaced when it turns gray.
Additional filters can be procured from your dealer.

2. Only authorized personnel shall make repairs to this product.

CAUTION: The nebulizer may be damaged if used with a dirty filter or if the filter is replaced by other material such as cotton. Do NOT operate without a filter.

Technical data

Particle Size: **MMAD * approximately 3 μm

Appropriate Medication Quantities: 2ml minimum -6 maximum

Nebulization rate: 0.33ml/min (by weight loss)

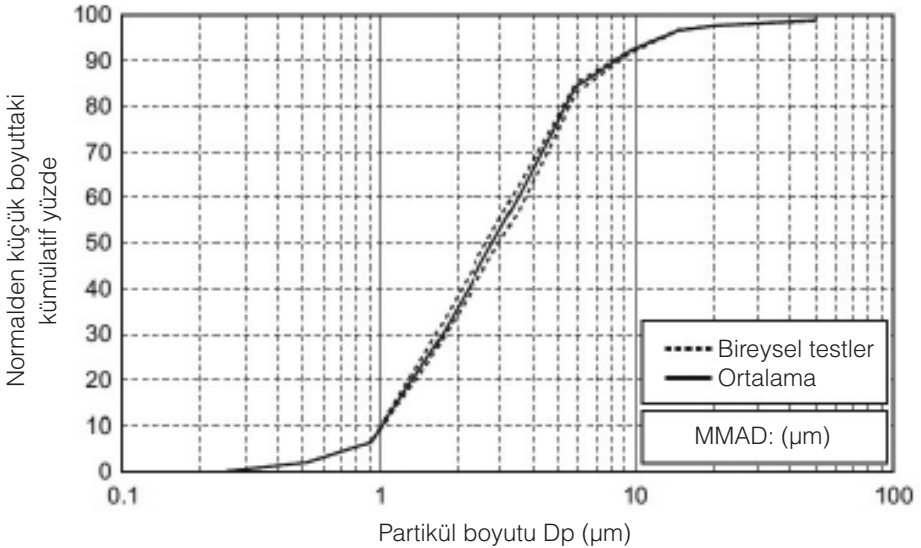
Aerosol Output: **0.33 ml (2ml, 1%NaF)

Aerosol Output Rate: **0.06ml/min (2ml, 1%NaF)

Compressor nebulizer the particle size distribution curve as well:

Result of cascade impactor **measurements for particle size with Compressor nebulizer and nebulizer kits

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MMAD = Mass Median Aerodynamic Diameter

Disposal

Once the product lift has ended or its components do not work anymore. Their disposal should be carried out according to the current regulations

Guidance and manufacture's declaration – electromagnetic emission		
This is intended for use in the electromagnetic environment specified below. The customer of the user should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	Use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. Is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emission CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacture's declaration – electromagnetic immunity			
This is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/ burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	±1 kV differential mode	Mains power quality should be that of a typical commercial or hospital environment.

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	<5% U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the nebulizer requires continued operation during power mains interruptions, it is recommended that the nebulizer be powered from an uninterruptible power supply or a battery.
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE U_T is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacture's declaration – electromagnetic immunity

This is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
---------------	----------------------	------------------	--

<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>$3 V_{rms}$ 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2.5 GHz</p>	<p>3 Vrms</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the nebulizer, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2,3\sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p>			
<p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

- a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the nebulizer is used exceeds the applicable RF compliance level above, the nebulizer should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the nebulizer
- b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between

portable and mobile RF communications equipment and the nebulizer.

This is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the nebulizer as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter(m)		
	150 KHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and

reflection from structures, objects and people.

Troubleshooting

Check the following if your unit should fail during operation. You can also refer to the pages of this manual for complete instructions.

Problem	Cause	Remedy
Nothing happens when the power switch is pressed.	Is the AC adapter connected correctly to the an electrical outlet and the compressor?	Check that the plug is inserted in an electrical outlet. Unplug then reinsert the plug if necessary.
No nebulisation or low nebulisation rate,when the power is on.	Is there medication in the medication tank?	Add the correct amount of medication to the medication tank.
	Is there too much/little medication in the medication tank?	
	Is the vaporiser head missing and not assembled correctly?	Attach the vaporiser head correctly.
	Is the nebulizer kit assembled correctly?	Assemble the nebulizer kit correctly.
	Is the nozzle blocked?	Make sure that the nozzle is free of blockages.
	Is the nebulizer kit tilted at a sharp angle?	Make sure that the nebuliser kit is not tilted at an angle of more than 45 degrees.
	Is the air tube connected correctly?	Make sure that the air tube is correctly connected to the compressor and nebuliser kit.
	Is the air tube folded or damaged?	Make sure that the air tube does not contain kinks.
	Is the air tube blocked?	Make sure that the air tube is free of blockages.
Is the air filter dirty?	Replace the air filterwith a new one.	
The device is abnormally loud.	Is the air filter cover attached correctly?	Attach the air filter cover correctly.
The device is very hot.	Is the compressor covered?	Do not cover the compressor with any type of cover during use.
	Is the device operating continuously over 20 minutes?	Limit use to 20 minutes at a time, and allow a 40 minutes interval before using the device again.

Note: If the suggested remedy does not solve the problem, do not try to repair the device- no parts of the unit are user serviceable.

Return the unit to an authorized retail outlet or distributor.

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Electrical Requirements	120V/60Hz, or 230 V / 50Hz
Power Consumption	Below 65W or 60W
Rated Current	Below 1.5A or 0.70A
Sound Pressure Level	58 dB(A)
Max.Comp.Pressure	35 Psi to 50psi (241 kpa to 345 kpa)
Neb.Operating.Pressure	9~ 16Psi (62kpa to 110kpa)
Liter Flow Range	5-8 L/Min
Nebulization Rate:	≥0.25 ml / Min
Particle size	3 microns
Maximum nebulizer solution capacity	6ml
Maximum residual medicine volume	0.5 ml
MMAD:	4um
Average Mobilization Rate:	Min 0.32ml / Min
Remain Amount:	0.15ml
Operating Temperature Range°C	10°Cto 40°C(50°F to 104°F)
Operation Humidity Range:	10%~95%RH
Storage Temperature Range°C	-20°Cto 70°C(-4°F to 158°F)
Storage Humidity Range:	10%~95%RH
Dimension(LxWxH):	Length 280mm x Width 165mm x Height 90mm
Products Weight:	1.8 kgs

Standard Accessories:	Compressor Nebulizer Machine : 1pcs
	Adult Mask: 1pcs
	Air Filter:1pcs
	Durable Air Tube:1pcs
	Medical Cup 6CC:1pcs
	Mouth Piece: 1pcs
	Pièce de nez: 1pcs

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